INSTRUCTION FOR USE

INTENDED FOR USE

This product to be used with insulin injector is a single use sterile insulin pen needle to be used for injecting insulin to patient with diabetes.

USAGE METHOD

1. Remove pen needle seal
2. Click pen needle onto pen, by pressing straight down using the outer protective cap (Do not click on at an angle).
3. Remove the outer protective cap by pulling straight off save it.
4. Remove the inner protective cap.
5. Penetrate skin quickly and inject slowly.
6. Carefully replace the outer protective cap (if selfinjection only).
7. Unscrew the pen needle and safely dispose of it according to your physician's or pharmacist' advice.

⚠ WARNINGS

Avoid infection

1. Do not use the pen needle if the seal is damaged or has been opened someone other than you.
2. Do not use the pen needle after the expiration date on the packaging has expired.
3. Before each injection: clean the injection site thoroughly according to your physician's instruction.
4. Use each pen needle only once.
5. Do not re-use, clean and/or re-sterilize pen needles.
6. Carefully replace the outer protective cap on the used pen needle and dispose of it immediately after self-injection, follow your physician's or pharmacist's advice for proper disposal in your area.
7. If these instructions are not followed, there is a risk of infection.
8. Seek medical attention if you have any health concern. Never Handle someone else's pen.

Get the correct dosage

1. Consult your physician for the right needle length for you.
2. Prime your pen according to your pen's instruction for use before each injection to ensure that the needle allows free flow of medication.
3. Inject slowly according to your physician's advice and pen's instructions for use.
4. Check after each injection whether medication drips from the pen needle, pen, or injection site.
5. If the medication drips, follow your physician's instructions regarding the handling of potential under-dosage.

A SAFE AND COMFORTABLE INJECTION

1. Change the injection site regularly in accordance with your physician's advice. If this is not done, tissue hardening may result.
2. Inject medication at room temperature.
3. Make sure no air bubbles remain in the pen before injection.
4. Wait until topical alcohol (if used) has evaporated completely before injection.
5. Keep muscles in the injection area relaxed when injecting.
6. Penetrate the skin quickly.
7. Do not change the direction of the needle during insertion into or withdrawal from the skin.

COMPATIBLE INJECTORS

PEN NAME	MANUFACTURER
ReliOn InnoLet	Novo Nordisk
Novopen 1,5ml	Novo Nordisk
Novopen 3ml	Novo Nordisk
Novolog FlexPen	Novo Nordisk
Humulin Pen	Eli Lilly
Humalog Pen	Eli Lilly
HumaPen	Eli Lilly
SoloStar	Sanofi
BD Pen 1,5ml	Beckon Dickinson
Auto Pen	Owen Mumford
BerliPen	Berlin Chemie
Omnican Pen	B.Braun

Description of the symbol on the Label

-  Symbol for "Manufacturer"
-  Symbol for "authorized representative in the EC"
-  Symbol for "use by"
-  Symbol for "manufacture Date"

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